

ToeUP™

Textile foot drop orthosis

A new foot drop orthosis from Allard!

ToeUP is a textile ankle foot orthosis designed for individuals with a mild form of foot drop. The textile surface makes the orthosis comfortable and soft against the ankle, while it, together with the elastic cords, provides stability and support for a lift of the forefoot during walking.

- Provides a lift of the forefoot during walking
- The support is regulated by the placement of hooks and elastic cords
- Soft and flexible, for outdoor and indoor use
- The D-ring can hold the elastic bands together for a more discreet silhouette



Support and comfort for mild foot drop

ToeUP™ features a low-profile textile ankle section that fastens above the malleoli. The ankle section is connected using an elastic cord with hooks, which are secured beneath the lacing or within the lacing holes on the shoe. This mechanism provides elevation for the forefoot during ambulation. The level of elevation can be adjusted by altering the placement of the hooks or the strap on the ankle section.

The supplementary D-ring can be used to hold the elastic bands together and allows for individualized positioning.

ToeUP™ offers an optional foot part component for indoor use, or with unlaced shoes such as sandals.



The D-ring fastens the straps together.



Foot part for indoor use or with unlaced shoes.



Item no.	Model	Size	L/R	Circumference of the ankle (1)	Circumference around foot (2)
285000010	ToeUP	X-Small	Bilat.	13-17 cm	-
285000011	ToeUP	Small	Bilat.	17-21 cm	-
285000012	ToeUP	Medium	Bilat.	21-25 cm	-
285000013	ToeUP	Large	Bilat.	25-29 cm	-
285011011	ToeUP Foot part	Small	Left	-	17-21 cm
285011012	ToeUP Foot part	Medium	Left	-	21-25 cm
285011013	ToeUP Foot part	Large	Left	-	25-29 cm
285012011	ToeUP Foot part	Small	Right	-	17-21 cm
285012012	ToeUP Foot part	Medium	Right	-	21-25 cm
285012013	ToeUP Foot part	Large	Right	-	25-29 cm
285020011	ToeUP Elastic Cord	X-Small - Small	-	-	-
285020013	ToeUP Elastic Cord	Medium - Large	-	-	-
285030000	ToeUP Hooks	-	-	-	-
285040000	ToeUP D-ring	-	-	-	-
285050000	Under lace hook	-	-	-	-
285060010	ToeUP Ankle Part	X-Small	Bilat.	13-17 cm	-
285060011	ToeUP Ankle Part	Small	Bilat.	17-21 cm	-
285060012	ToeUP Ankle Part	Medium	Bilat.	21-25 cm	-
285060013	ToeUP Ankle Part	Large	Bilat.	25-29 cm	-



Scan or click
for more
information!





Support for Better Life

Everyone should be able to live their life to the fullest, regardless of their mobility challenges. With innovative solutions developed in close collaboration with healthcare professionals and patients, we strive to provide Support for Better Life.

allard

Allard Support UK Ltd.

Coes Road, Dundalk, Co. Louth, A91 E868 | +353 42 932 8177 | info@allarduk.co.uk | allarduk.co.uk