allard

Support for better life!

Instructions for Use

Selection®
Open Wrist Children
Art. No. 35306

Instructions For Use

EN	SELECTION® OPEN WRIST CHILDREN	p. 4
SV	SELECTION® OPEN WRIST CHILDREN	p. 5
DA	SELECTION® OPEN WRIST CHILDREN	p. 6
NO	SELECTION® OPEN WRIST CHILDREN	p. 7
FI	SELECTION® OPEN WRIST CHILDREN	p. 8
ES	SELECTION® OPEN WRIST CHILDREN	p. 9
IT	SELECTION® OPEN WRIST CHILDREN	p. 10
NL	SELECTION® OPEN WRIST CHILDREN	p. 11
DE	SELECTION® OPEN WRIST CHILDREN	p. 12
FR	SELECTION® OPEN WRIST CHILDREN	p. 13

Intended use

To immobilize CMC and MCP 1 joint and wrist in order to reduce pain, to guide the wrist and an adducted thumb into a neutral position and to support a wrist after trauma/post operatively if needed. Intended for one person use.

Indications

CP, Rheumatoid arthritis fi thumb is adducted, post trauma. Single Patient-multiple use.

Contraindications

No known.

Adjustments

Fitter should make sure the stay is properly adjusted. This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This product is made from polyester, Lycra and Nylon. Spoons are made from aluminum. This product is not flame retardant.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. Remove stay before laundering. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal


Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

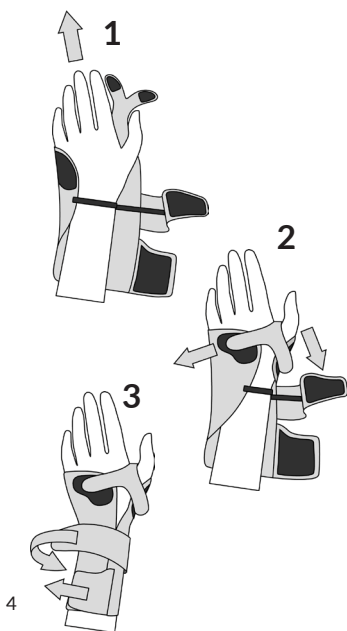**Instructions for use**

1. Check that the support is complete and without damage.
2. Bend the stay to fit the shape of the wrist (preferably with the stay in the pocket).
3. Position the hand in the open brace (Image 1).
4. Close the strap around the thumb and the strap on the dorsal side of the hand (Image 2).
5. Close the circular strap and the strap on the arm (Image 3).
6. Adjust the straps around thumb into correct position.
7. Adjust all straps for comfort.

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