allard
Support for better life!

Instructions for Use

Selection®
Short wrist
Art. No. 35222/35223

Instructions For Use

| | | |
|----|------------------------|-------|
| EN | SELECTION® SHORT WRIST | p. 4 |
| SV | SELECTION® SHORT WRIST | p. 5 |
| DA | SELECTION® SHORT WRIST | p. 6 |
| FI | SELECTION® SHORT WRIST | p. 7 |
| NO | SELECTION® SHORT WRIST | p. 8 |
| IT | SELECTION® SHORT WRIST | p. 9 |
| DE | SELECTION® SHORT WRIST | p. 10 |
| NL | SELECTION® SHORT WRIST | p. 11 |
| FR | SELECTION® SHORT WRIST | p.12 |
| ES | SELECTION® SHORT WRIST | p. 13 |
| JA | SELECTION® SHORT WRIST | p. 14 |

Carefully read this instruction sheet before use!

Intended use/Indications

Supports and relieves the wrist after injuries, post surgery and casting, hypermobility, rheumatic problems, chronic pain, arthritis or overload symptoms. Can also be used in preventive purpose during heavy or repetitive movements. Single Patient-multiple use.

Contraindications

No known.

Adjustments

This products should only be altered by medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This iproduct is made from polyester, nylon and elasthan. This product is not flame retardant.

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F) gentle cycle. User a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal


Disposal in accordance with local legislation.

Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

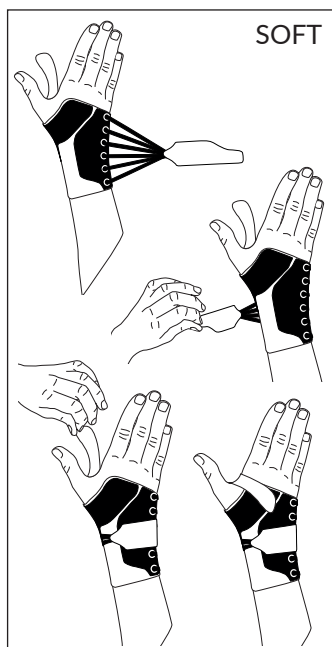Instructions for use

1. Check that the support is complete and without damage.
2. Open the Velcro straps and release the lacing.
3. Pull the orthosis on like a glove.
4. Secure the strap around the wrist.
5. Secure the thumb strap.
6. Adjust for comfort.

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