


Patient guide


S.O.T thumb orthosis

FITTING


To ensure optimal function it is important that the cover is well adjusted. Attach the cover in sequences, leave the thumb section (1) attached from the start. Then attach 2 and 3, and if needed adjust 1. Finally adjust and make sure it fits comfortably.

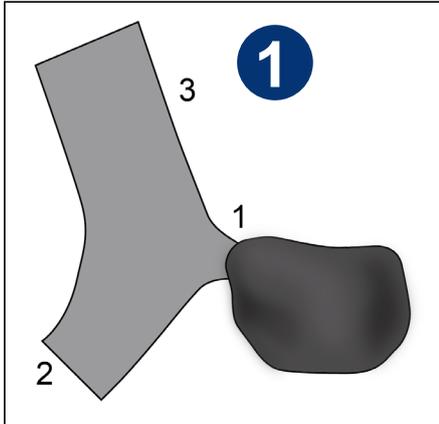

Open the wide straps 2 & 3 and leave strap 1 at its position.

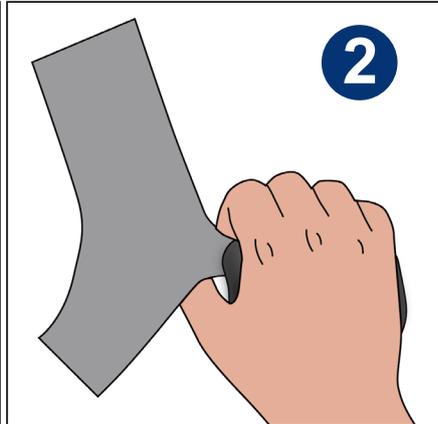



Put the hand on the support.

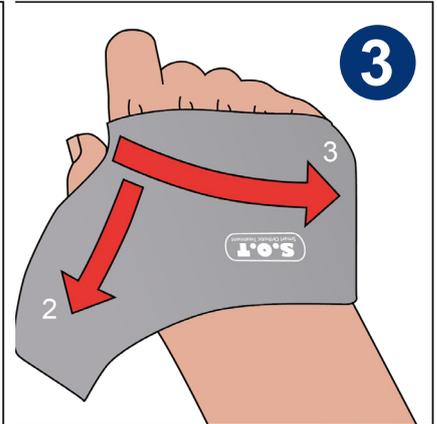

Pull the strap over the hand and attach strap 3 and then strap 2 on the Velcro on the inside of the support. Adjust strap 1 and 2 for comfort.

PUT ON

TAKE OFF

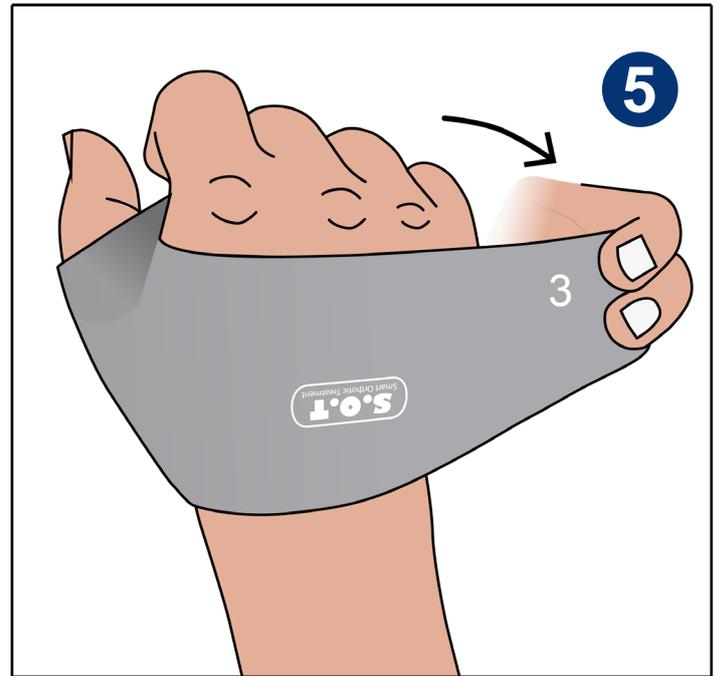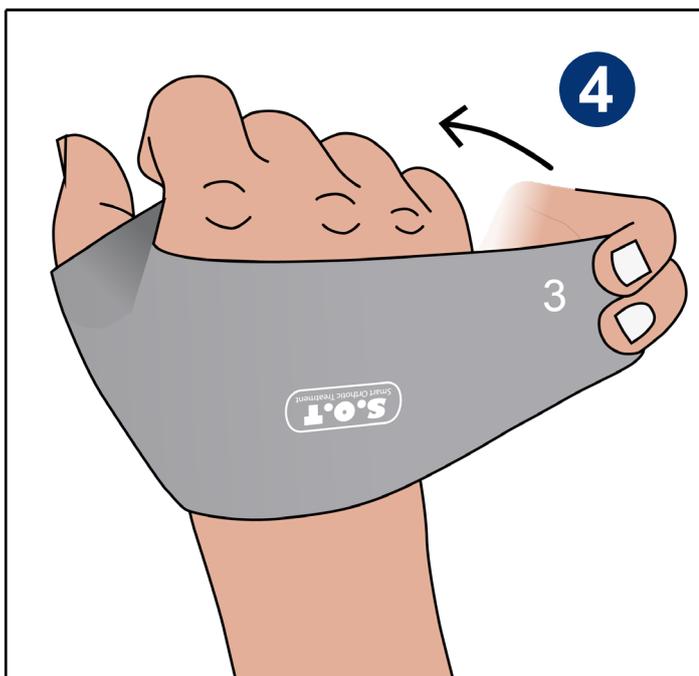

For donning and doffing, after initial fitting, only open strap on the little finger side. Strap 3.

When to use the product:

Good to think about:

WASHING INSTRUCTION: Wash the complete product with the cover attached.
Use a washing bag. Machine wash in 40°C (104°F) gentle cycle. Dry flat.
Do not tumble dry or dry clean.

If any problem occurs please contact:

Name: _____

Telephone: _____

CampScandinavia_april_2023©