

Allard UK is now assembling COMBO™

Saving you time in the workshop!

COMBO™ Assembly Order Forms can be downloaded from our website.



LOW PROFILE AND FUNCTIONAL ORTHOTIC SOLUTION

FOR MANAGEMENT OF GENU-RECURVATUM
COMBINED WITH FOOTDROP.

ToeOFF® has during the years proven to be an effective gait rehabilitation orthosis for management of footdrop and ankle instability. It has helped hundreds of thousands of people around the world walk with a closer to normal gait pattern, often being able to return to activities that significantly enhance quality of life. Often, footdrop is accompanied by genu-recurvatum. Combo is a lightweight, removable knee brace attachment for ToeOFF® (or BlueRocker®) that creates an innovative orthotic solution for management of footdrop combined with genu-recurvatum.

COMBO™
an ALLARD product

When treating patients that suffer from different types of nerve and muscle disorders like Stroke, Post-polio and CMT often you see remaining deficits like foot drop. As the muscles get weaker, other problem can occur. One of these can be Genu Recurvatum. Malposition, excessive laxity, instability and muscle shortening can increase the stress and overload can turn into pain. In longer term gait velocity and step length can be affected, and asymmetry can cause other injuries in the body.


- Uprights easily shaped and adjusted to optimum height for patient
- Modular components allow you to accommodate variances in thigh and calf circumferences
- Low profile
- Lightweight
- Easy to don and doff
- Easily removed when wearer is relaxing (watching TV, etc.)



COMBO™ is fitted individually for each patient. The different components can be combined to accommodate patient's size and length.

- Size of thigh cuff is determined depending on how high on the thigh it is placed. However it cannot be closer to the Tibia Plateau than 10 cm (4 inches).
- Size of Tibia connector is determined by instead of from what size the ToeOFF® or BlueROCKER® will be.
- Size of the Hyperextension adjustment is determined from knee width (M-L).
- All 5 product codes need to be ordered for a complete orthosis.
- COMBO™ cannot be used for patients above 150 kg and 2 meters.

COMBO™
an ALLARD product

Item No.	Description	Size	Left/Right	Circumference*	M-L*
28780 0000	Joints with uprights	One size			
28781 1011	Thigh Cuff	Small	Left	30 - 40 cm	11 - 15 cm
28781 1012	Thigh Cuff	Medium	Left	38 - 50 cm	13 - 15 cm
28781 1013	Thigh Cuff	Large	Left	48 - 60 cm	15 - 19 cm
28781 1014	Thigh Cuff	XLarge	Left	58 - 70 cm	17 - 21 cm
28781 2011	Thigh Cuff	Small	Right	30 - 40 cm	11 - 15 cm
28781 2012	Thigh Cuff	Medium	Right	38 - 50 cm	13 - 15 cm
28781 2013	Thigh Cuff	Large	Right	48 - 60 cm	15 - 19 cm
28781 2014	Thigh Cuff	XLarge	Right	58 - 70 cm	17 - 21 cm
28782 0011	Hyperextension Adjustment	Small			70 - 105 mm**
28782 0012	Hyperextension Adjustment	Medium			95 - 130 mm**
28782 0013	Hyperextension Adjustment	Large			125 - 160 mm**
28783 0011	SoftKIT for 28781 set of 2	Small			
28783 0012	SoftKIT for 28781 set of 2	Medium			
28783 0013	SoftKIT for 28781 set of 2	Large			
28783 0014	SoftKIT for 28781 set of 2	XLarge			
28784 0011	Tibia Connector	Small	 Choose size same as ToeOFF®/BlueROCKER®		
28784 0012	Tibia Connector	Medium			
28784 0013	Tibia Connector	Large			