# BOSTON HIP ABDUCTION BRACE - Teknisk information (SE)

# BOSTON HIP ABDUCTION BRACE - Teknisk information (DK)

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## ENG

The Boston Hip Abduction brace is a lightweight paediatric orthosis with a frame that is manufactured from High Density Polyethylene and a closed cell foam liner. The orthosis is designed to maintain the legs in abduction and flexion or "frog leg" position. This ensures that the hip joint is held in the optimum position to encourage the correct development of the hip joint. It is available in five sizes.

### Indications

The Boston Hip Abduction brace is used for patients with the following conditions:

*Congenital Hip Dislocation*

*Hip Dysplasia*

*Cerebral Palsy*

### Measurement for Hip Brace

To determine the correct size lie the patient supine in the "frog" position. Measure the pelvic circumference, next measure the distance between the backs of both calves. Check these measurements with the brace measurement chart. If the patient falls between sizes use the larger size of brace and trim the length of the thigh section. This will increase the life of the brace.

### Fitting the Hip Brace

The Boston Hip Abduction brace should be worn over the top of clothes. Stretchy pants or leggings are best and thickness and materials should be appropriate to the season.


*There are two methods of putting the brace on, use whichever method you find most comfortable.*

The first option is to lay the child on their back on a flat surface (bed/changing mat/floor) and lift up the legs and slide the brace underneath their bottom. The other method is to place the brace on a flat surface with the straps open and place the child into the brace.

The child should sit roughly in the middle of the brace. Position each thigh into the brace. The foam padding should wrap around the thigh and tuck tightly into the back of the brace. (If the foam is too long it can be trimmed with a pair of scissors). The thigh straps can now be fastened firmly. The waist strap should be fastened with about three fingers under the strap whilst doing up the Velcro. This allows some room under the waist strap so that it is not too tight when sitting.

### Care of Hip Abduction Brace

The Hip Abduction Brace can be cleaned with soap and water and dried with a towel or left to dry naturally.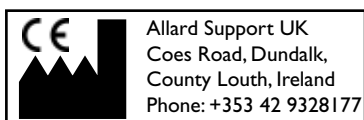

Distributed in Scandinavia by:  
Camp Scandinavia AB  
Karbingatan 38, 254 67 Helsingborg,  
Sweden  
Phone: 46 42 25 27 01