

Elements Body – Vest & Body



Elements Body Vest & Body Suit is a textile orthosis that provide continuous proprioceptive feedback over the trunk and upper body. The vest version may be particularly suitable when support is required for postural control and trunk stability, with the aim of improving posture and coordination in daily activities.

The Elements Body orthosis is made of Lycra elastic fabric and constructed from multiple panels in a design that provides pressure and resistance to enhance posture and stability. By increasing deep sensory input, also referred to as proprioception, fine motor skills as well as functional performance may also be improved.

The orthosis is ordered and manufactured according to individual measurements and is available with or without sleeves. Reinforcements and zippers can be added as needed for functional and usability needs. The design and configuration are determined by the prescribing clinician.

Advised minimum wearing time for Elements Body is 6 hours a day. At the beginning of treatment, Elements Body should be worn for shorter periods of time, and gradually increased until the goal of 6 hours per day is reached. The orthosis should not be used for more than 8–10 hours per day. Elements Body should not be worn through the night when patient is sleeping.

Recommended Range Of Application

Cerebral Palsy: hypotonia, spasticity, ataxia

MS, Stroke & Acquired Brain Injury: spasticity, ataxia

Muscular Dystrophy, Acquired Spinal Cord Injury and Down's Syndrome: hypotonia

Ataxia: spastic ataxia

Item No.	Model
393000000	Vest Without Sleeves
393060012	Body with ¾ sleeve
393010011	Vest With Short Sleeves
393010012	Vest with ¾ sleeve
393010013	Vest With Long Sleeves
393050000	Body Suit Without Sleeves
393060011	Body Suit with Short Sleeves
393060013	Body Suit With Long Sleeves
393500000	Extra Reinforcement Small
393510000	Extra Reinforcement Large
393600000	First Free of Charge Alteration
393610000	Small Alteration
393630000	Large Alteration