

KiddieFLOW™

Unlike KiddieGAIT®, KiddieFLOW™ is more flexible in the heel and toe area of the footplate and allows a greater range of motion. KiddieFLOW™ is based on the same principles as our adult FLOW models and retains the same geometry as KiddieGAIT® and KiddieROCKER®. The orthosis supports the dorsiflexion of the foot in the swing phase, provides better control of foot positioning in the late swing phase, which in turn improves stability in the stance phase. At the same time, the softer foot blade provides easier roll off at the end of the stance phase.

For a combination of increased A / P and M / L control, KiddieFLOW™ can, in addition to our other children’s models, also be combined with individual insoles or SMO.

Recommended Range Of Application

Footdrop. KiddieFLOW™ is designed to support footdrop, gait deviations secondary to proprioceptive deficit, unstable gait or low-tone gait, toe-walking with no midfoot collapse, Spina Bifida, Cerebral Palsy and Muscular Dystrophy.

Contraindications

KiddieFLOW™ should not be used when patients present with: Lacking ROM towards dorsiflexion (need at least 5o dorsiflexion past neutral), Very rigid foot structure, Quadriceps spasticity, Fixed postural Genu Valgum or Genu Varum, Fixed postural Pes Valgus or Pes Varus. KiddieFLOW™ should not be selected for children present with crouch gait.



Item No.	Model	Size	L/R	Footplate Length	Height
288911011	KiddieFLOW™	Small	Left	16 cm	22 cm
288911012	KiddieFLOW™	Medium	Left	18 cm	25,7 cm
288911013	KiddieFLOW™	Large	Left	20 cm	29,5 cm
288911014	KiddieFLOW™	X-Large	Left	21 cm	31,5 cm
288912011	KiddieFLOW™	Small	Right	16 cm	22 cm
288912012	KiddieFLOW™	Medium	Right	18 cm	25,7 cm
288912013	KiddieFLOW™	Large	Right	20 cm	29,5 cm
288912014	KiddieFLOW™	X-Large	Right	21 cm	31,5 cm